



TCU™ WALL BRACKET MOUNTING INSTRUCTIONS
PART #150-W

SELECT A MOUNTING LOCATION THAT IS:

- 1. CONVENIENT FOR USE**
- 2. NEAR AN ELECTRICAL OUTLET**
- 3. AT A POINT ON THE WALL WHERE A SECURE MOUNT MAY BE ACHIEVED (STUD)**

MAKING SURE THAT THE "TOP" LABEL ON THE REAR OF THE BRACKET IS AT THE TOP POSITION, SECURE THE WALL BRACKET TO THE WALL USING THE SUPPLIED SCREWS.

MATCH THE FOUR RECESSED HOLES ON THE REAR OF THE TCU TO THE FOUR POSTS ON THE WALL BRACKET AND GENTLY SLIDE THE TCU ONTO THE POSTS. IF YOU NOTICE THE RUBBER GASKETS SLIDING OUT OF POSITION, STOP AND RETURN THE GASKETS TO THEIR PROPER LOCATION AND START AGAIN.

THE TCU IS IN PROPER POSITION WHEN:

- 1. THE TWO BOTTOM RUBBER FEET ON THE REAR OF THE UNIT TOUCH THE MOUNTING BRACKET, AND**
- 2. THE TWO TOP RUBBER FEET ON THE REAR OF THE UNIT ARE APPROXIMATELY 3/8" FROM THE BRACKET. DO NOT ATTEMPT TO FORCE THE TOP OF THE TCU CLOSER THAN 3/8" FROM THE BRACKET.**

TO REMOVE THE TCU FROM THE BRACKET, GENTLY ROCK THE UNIT FROM SIDE TO SIDE AND PULL BACK UNTIL THE TCU IS FREE FROM THE POSTS. UPON REMOVAL, THE RUBBER GASKETS MAY SLIDE OUT OF POSITION - BE SURE TO LOCATE ALL FOUR AND RETURN THEM TO THEIR PROPER LOCATIONS.

81-0020(05/01) REV-A